Tiger Dental

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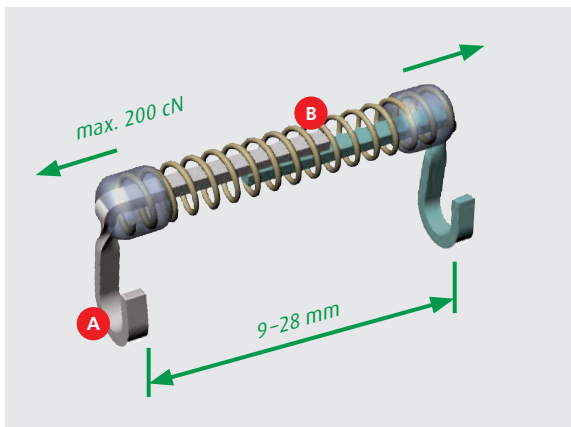
Achtung

English

1 Function and parts of the piggy back spring

A Hook

B Pressure spring

| Size | Colour | Function |
|------|-------------|------------------------------------|
| F1 | Transparent | Bypass of a front tooth bracket |
| B1 | Orange | Bypass of a lateral tooth bracket |
| F2 | Red | Bypass of 2 front tooth brackets |
| B2 | Blue | Bypass of 2 lateral tooth brackets |
| U3 | Green | Bypass of 3 brackets (universal) |
| CC | Black | Eye tooth-eye tooth displacement |

Dear doctor, we are honored that you have chosen this innovative new product from Tiger Dental GmbH.

To ensure the safe and effective use of this product, you must carefully read and adhere to this user manual.

The user manual covers all necessary safety instructions but cannot describe all possible circumstances. For this reason, you may find further user notices online at tigerdental.com under the “Know-how” link. Our customer service is available for any additional questions.

The continual development of our products indicates our company’s commitment to innovation. Accordingly, after several clinical uses, we recommend you carefully review the accompanying user manual (this can be found online under the “Downloads” link at tigerdental.com).

1. Manufacturer

Tiger Dental GmbH, Belruptstraße 59, 6900 Bregenz, Austria

2. Product description

The piggy back spring is a pressure spring device specifically constructed (Fig. 1) to move teeth apart during orthodontic treatment. The piggy back spring is placed directly in the area of the crowding whilst bypassing one or more brackets. This leads to pressure on the outer teeth in the area of crowding in the levelling phase.

In contrast to traditional pressure springs, the piggy back spring does not support the levelling arch during outward rotation and insertion of malpositioned teeth. It is attached in a few seconds without removing the treatment arch. Brackets that are already in place do not have to be removed. Six different sizes enable space to be gained for individual malpositions and eye tooth to eye tooth distance can be extended. With full compression, the force provided is about 200 cN (progressively reducing).

3. Applications

Indications:

- Gaining space with front teeth crowding
- Opening a gap in the area of the lateral teeth
- Outward rotation of teeth

- Straightening of lingually tilted UK eye teeth (CC)
- Mesialisation of anterior lateral teeth
- Distalisation of posterior lateral teeth

4. Instructions for use

Installation in the mouth

The piggy back spring is pre-tensioned with an elastic band for easy installation. A piece of dental floss should be firmly knotted to the elastic band for secure installation in the mouth. The other end of the floss is then held by an assistant or the patient. The right size (length) of piggy back spring is measured between the relevant brackets surrounding 1, 2, 3 or 4 teeth. The two hooks are then attached to the treatment arch. After the hooks have been closed, both sides of the elastic band are separated and all of the dental floss is removed. This activates the piggy back spring. For the arch to slide better, the ligatures on both sides distally to the piggy back spring are loosened. This is ideally carried out by pulling up the bracket ligature with a dental probe (Fig. 2-4).

Removal

To remove the piggy back spring, the closed hooks are separated with a side cutter or curved up with grape scissors. The piggy back spring should again be secured during this procedure with a piece of dental floss (Fig. 6).

5. Warnings

1. Incorrect manipulation with pliers can damage the safety heads which join and secure the spring element to the glide wires. If this happens on one side, the spiral spring is still attached on the other side to the glide wires. As both glide wires have been fixed to the treatment arch, these also may not be released into the oral cavity.
2. After the piggy back spring has been installed, the handler must make sure that it cannot come loose from the treatment arch (e.g. if the piggy back spring is pushed upwards).
3. The spring element should be removed from the mouth after about 30 days so that progress can be checked.
4. Regular short-term checks prevent overexpansion.

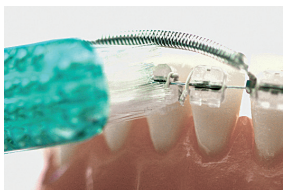
2 Fitting

3 Closing

4 Removing safeguard

5 Opening up for cleaning

6 Separating for removal

7 Before

8 After

6. Notices for Single-use Products

This product is nearly ready to use out of the box, but is not sterile and should not be used more than once. The reprocessing of this once used product (recycling) and its reapplication to the patient is strictly prohibited.

7. Safety instructions

This operating and installation manual contains tips and recommendations, but does not release you from personal responsibility! The use of this product is allowed only by orthodontists and dentists according to MDD 93/42/EEC. The user must comply with the provisions of the Medical Devices Directive and any other applicable laws and regulations.

 This product contains nickel.

8. References of Quality

Tiger Dental GmbH assures the user a flawless quality of its products. The content of this manual is based on experience. The user is responsible for the processing of the products themselves. Tiger Dental GmbH is not liable for erroneous results or processing by the user that occur in the absence of influence of Tiger Dental GmbH. Although this product is a mature product, improvements and changes can be made, which may not be documented in the present manual. Similarly, we are not liable for printing errors.

9. Contents

REF 10109, piggy back spring starter set, 1 item each size 1-6

REF 10255, piggy back spring range, 5 items each size 1-6

REF 10215, piggy back spring individual box F1, 5 items/pack

REF 10216, piggy back spring individual box B1, 5 items/pack

REF 10218, piggy back spring individual box F2, 5 items/pack

REF 10220, piggy back spring individual box B2, 5 items/pack

REF 10221, piggy back spring individual box U3, 5 items/pack

REF 10226, piggy back spring individual box CC, 5 items/pack

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